

### KIT CONTENTS


- 20 test cassettes
- 2 bottles of buffer
- 20 sterile nasopharyngeal swabs
- 20 extraction tubes and 20 filter tips
- 1 tube holder
- 1 instructions for use

Read the instructions  
for use carefully



1

### PREPARATION OF THE EXTRACTION BUFFER

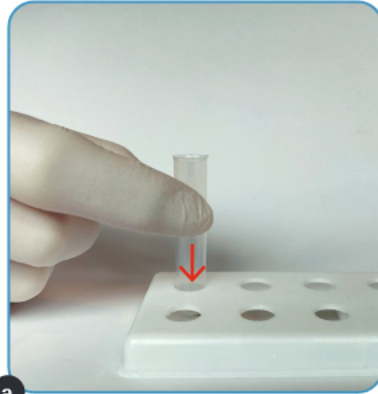

a

Insert the extraction tube into the  
bottom of the bracket




b

Add 16 drops of extraction buffer

2

### NASOPHARYNGEAL SWAB

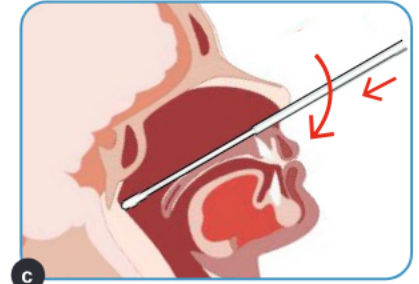


c

Insert the swab **horizontally** in relation  
to the face and push until it stops.  
Turn the swab several times for  
5 seconds.


3

### PREPARATION OF THE SAMPLE

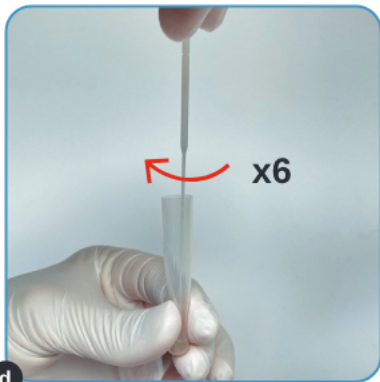


d

Dip the swab into the extraction tube and  
turn it at least 6 times while pressing its  
head against the bottom and sides of the  
extraction tube

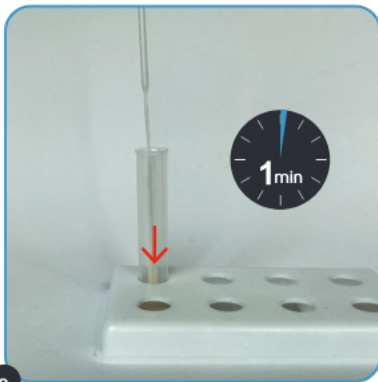

e

Leave the swab in the extraction  
tube for 1 minute

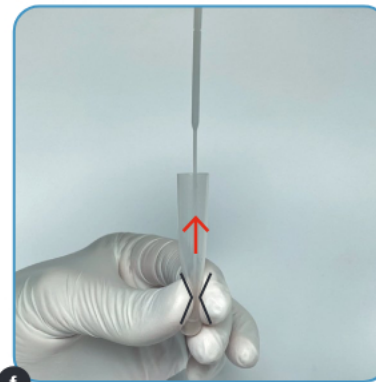

f

Remove the swab by pressing it  
against the walls of the extraction  
tube and discard it.



g

Insert the nozzle with the filter onto  
the extraction tube

**THE NOZZLE MUST BE FULLY  
INSERTED INTO THE TUBE**

CONTINUED ON BACK

4

### PROCEDURE


5

### READING




**WAIT 15 MINUTES BEFORE  
READING THE TEST**

Read the test in natural light or with sufficient illumination to distinguish bands of very low intensity

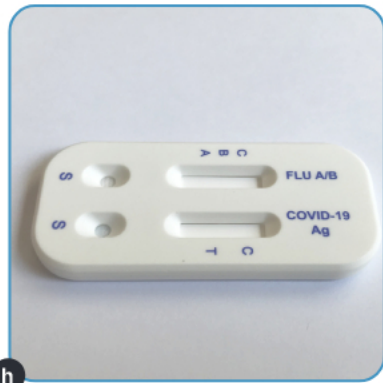


Remove the cassette from the bag and place it on a flat surface



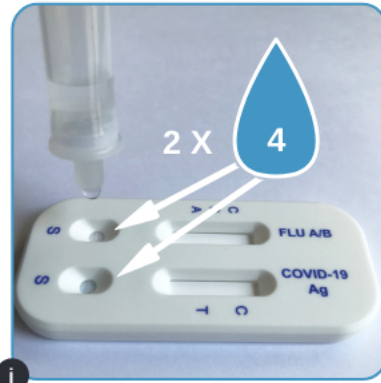


Place 4 drops of sample in each of the sample wells (S) of the cassette

**THE BOTTLE MUST BE  
KEPT UPRIGHT**

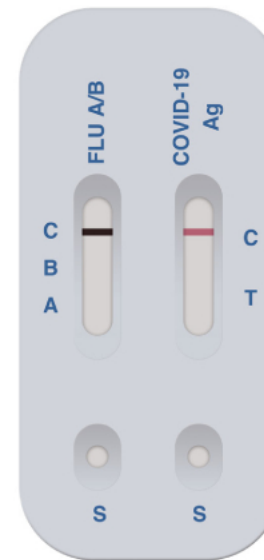

Flu and covid **NEGATIVE**

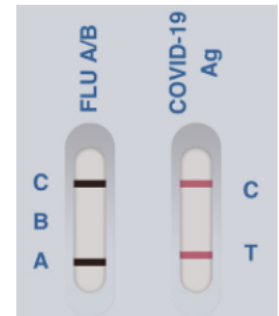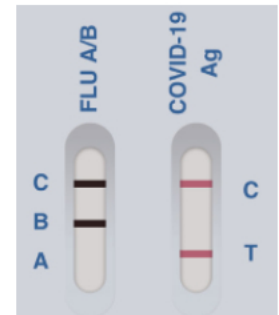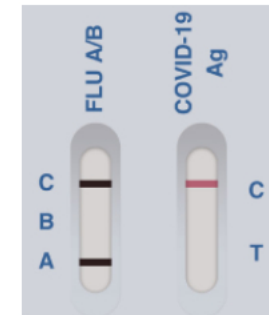

Some **POSITIVE** examples

The intensity of the bands may vary

CONTACT

+33 (0) 1 46 00 40 40  
[contact@aazlab.fr](mailto:contact@aazlab.fr)

**AAZ**   
Votre santé sous contrôle

43, rue de Bellevue | 92100 Boulogne-Billancourt

[www.aazlab.fr](http://www.aazlab.fr)