

### CONTENU DU KIT


- 20 test cassettes
- 20 sterile nasopharyngeal swabs
- 20 extraction tubes and 20 filter tips
- 1 tube holder
- 1 instructions for use

Read the instructions for use carefully



1

### PREPARATION OF THE EXTRACTION BUFFER

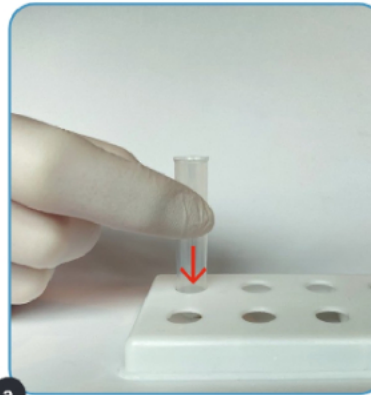


a

Remove the cover and Insert the extraction tube into the bottom of the bracket

2

### NASOPHARYNGEAL SWAB

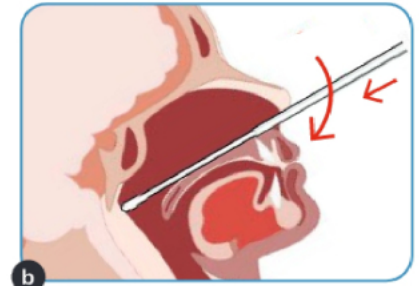


b

Insert the swab horizontally in relation to the face and push until it stops. Turn the swab several times for 5 seconds


3

### PREPARATION OF THE SAMPLE

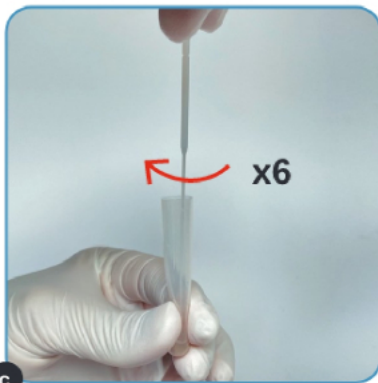


c

Dip the swab into the extraction tube and turn it at least 6 times while pressing its head against the bottom and sides of the extraction tube

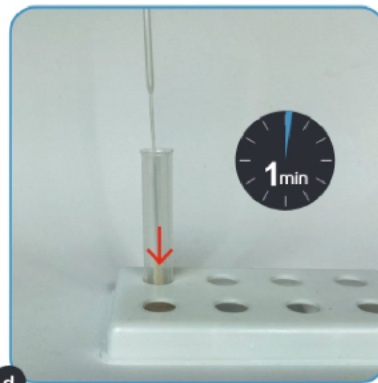

d

Leave the swab in the extraction tube for 1 minute

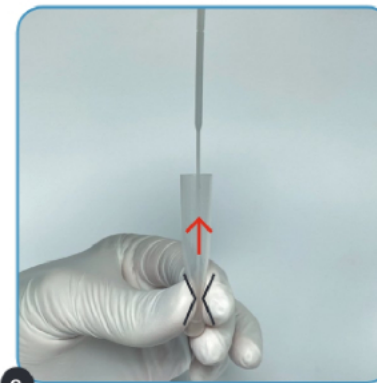

e

Remove the swab by pressing it against the walls of the extraction tube and discard it



f

Insert the nozzle with the filter onto the extraction tube

**THE NOZZLE MUST BE FULLY INSERTED INTO THE TUBE**

CONTINUED ON BACK

4

### PROCEDURE

5

### READING



**WAIT 15 MINUTES BEFORE  
READING THE TEST**

Read the test in natural light or with sufficient illumination to distinguish bands of very low intensity



g Remove the cassette from the bag and place it on a flat surface



h Place **4 drops** of sample in the sample well (S) of the cassette

**THE BOTTLE MUST BE  
KEPT UPRIGHT**



**NEGATIVE**



**POSITIVE**



The intensity of the bands may vary

AAZ.MKTVC.01-A (20241220)

### CONTACT

+33 (0) 1 46 00 40 40  
[contact@aazlab.fr](mailto:contact@aazlab.fr)

**AAZ**   
Votre santé sous contrôle

43, rue de Bellevue | 92100 Boulogne-Billancourt

**DEMONSTRATION VIDEO  
AVAILABLE ON :  
[aazlab.fr](https://aazlab.fr)**