




HYGIENIC CYSTITIS SCREENING KIT

STEP 1 Taking the sample

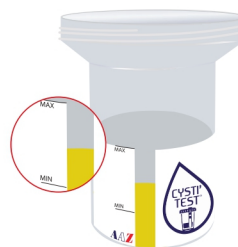

Take the sample,
the volume must be
between the **MIN** and
MAX dipstick marks

Remove excess urine
if the fill level exceeds
the MAX mark




Close the bottle

Place the bottle in
the plastic bag

STEP 2 Performing the test



Homogenise
the sample

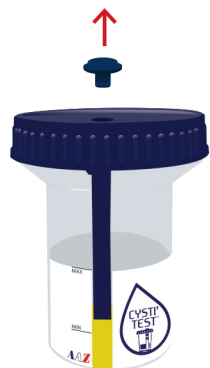


Remove the cap
from the lid



Arrows pointing
downwards

Patches visible
from the outside

Insert the strip
into the hole

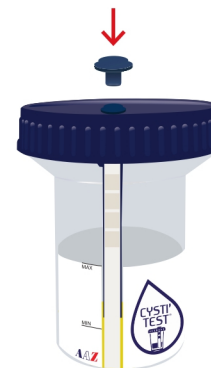


Replace the cap on the lid

Tilt the bottle for **1 second**
so that the strip is completely
soaked in water





STEP 3 Reading and interpreting the results

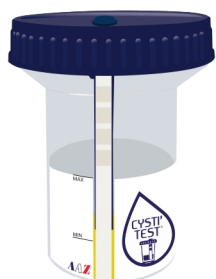

Read the results
at **2 minutes**

Use the colour chart
to render the diagnosis

Leukocytes
Nitrites
Blood
Proteins

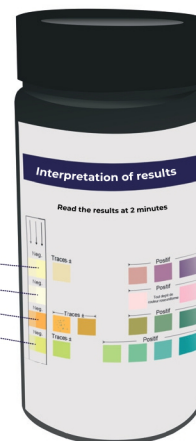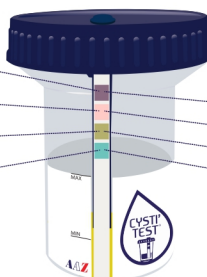

Read the results at 2 minutes

Leukocytes	Neg.	Traces ±	Positive		
Nitrites	Neg.		Positive		
		Traces ±	Positive		
Blood	Neg.				
Proteins	Neg.	Traces ±	Positive		